

# Oasis™

## DRY SUCTION WATER SEAL CHEST DRAIN



PRE-PACKAGED\*  
WATER

### SET UP

- Step 1. Fill Water Seal to 2cm Line**  
Add 45ml of sterile water or sterile saline via the blue suction port located on top of the drain. A pre-packaged water ampoule is provided on the back of each Oasis drain (*illustration 1*).  
Twist top off the ampoule and insert tip into suction port (*illustration 2*). Squeeze contents into water seal until fluid reaches 2cm fill line.
- Step 2. Connect Patient Tube to Patient**  
Connect chest drain to patient prior to initiating suction.
- Step 3. Connect Suction to Chest Drain**  
Attach suction line to blue suction port on top of chest drain (*illustration 3*).
- Step 4. Turn Suction Source On**  
Increase suction source vacuum to -80mmHg or higher. Suction regulator is preset to -20cmH<sub>2</sub>O. Adjust as required.



**100%  
Latex-Free**

**100%  
MRI Compatible**



illustration 1

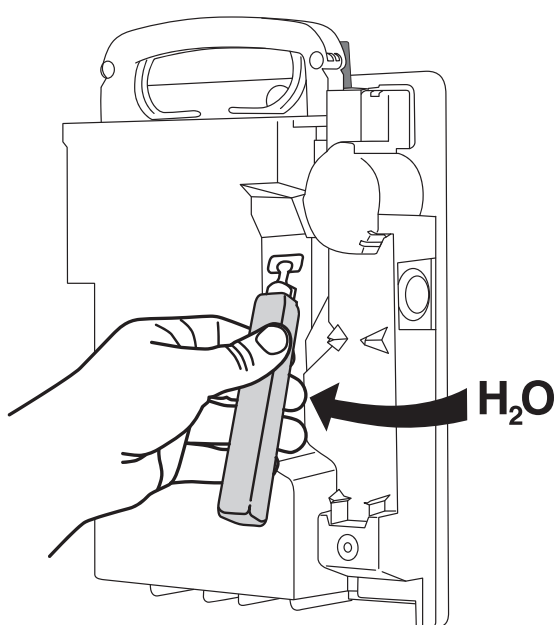


illustration 2


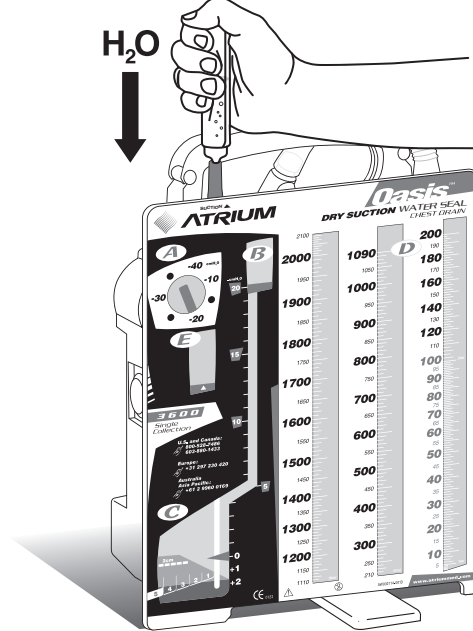


illustration 3



\*Patented

Have a question or need help in a hurry?  
Call Atrium toll free at 1-800-528-7486.

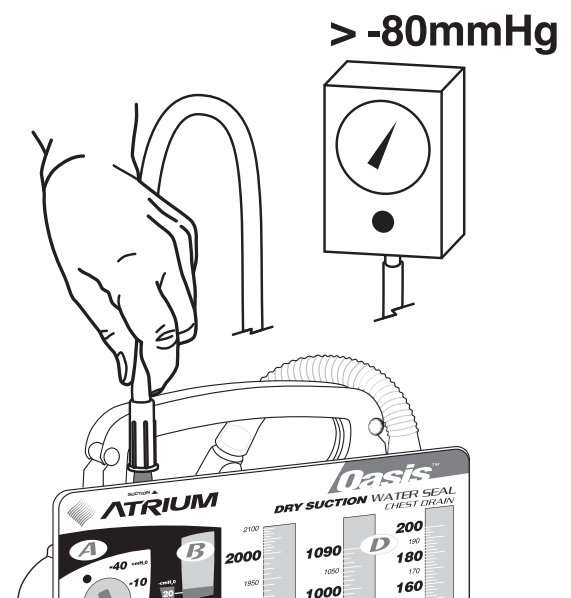

For additional education resources,  
visit us at [atriumuniversity.com](http://atriumuniversity.com).



**ATRIUM**